Seeing AI User Manual

Revision D

12-Dec-2024

EN

Seeing AI User Manual

Get Started

To use Seeing AI on your mobile device, please follow these instructions:

1. Make sure your device is up to date and running the latest version of iOS or Android.
2. Install Seeing AI from the Apple App Store from the following link: <https://aka.ms/SeeingAIiOSDownload> or from the Google Play Store from the following link: <https://aka.ms/SeeingAIAndroidDownload>
3. Open the app and follow the on-screen instructions to set up the device.
4. Once the device is set up, follow all in-app prompts and warnings.
5. If you experience any issues or have any questions, please contact the manufacturer for assistance, using the contact information listed in this manual.
6. It is important to follow all instructions and warnings provided by the manufacturer to ensure the safe and effective use of the device.

Use of this application is subject to the Seeing AI Terms of Use. It is your responsibility to confirm you understand and accept these terms and conditions as shown in the application.

Product Information

Product Name

Seeing AI

Seeing AI is a class I medical device in accordance with Regulation (EU) 2017/745 (for MDR).

Version

Available in the About Section of the Application

Date of Manufacture

Available in the About Section of the Application

Manufacturer

Microsoft Corporation
One Microsoft Way
Redmond, WA 98052
USA

European Union Authorized Representative

Emergo Europe B.V.
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

United Kingdom Responsible Person

Emergo Consulting (UK) Limited
c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

Swiss Authorized Representative

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

Regulatory Information

Basic UDI-DI: 08600098825SEEINGAIHOSDN

EMDN Code: Y99

iOS UDI-DI: 00860009882508

00860009882508

Android UDI-DI: 00860009882515

00860009882515

Important Safety Information

Intended Purpose

Seeing AI is a mobile app, for use with iOS and Android platforms, which provides a blind or otherwise visually impaired user with information about their visual environment by leveraging emerging technologies such as Artificial Intelligence (AI). Seeing AI is intended to augment an individual's existing awareness of their surroundings, by providing audio and haptic feedback about what has been detected.

Contraindications

Seeing AI is not designed to be and should not be used as a primary or exclusive technology by which persons who are blind or otherwise visually impaired learn about their environment. Seeing AI uses Artificial Intelligence (AI) to produce probabilistic outputs, which will vary in accuracy depending on where and how Seeing AI is used. Seeing AI does not detect, diagnose, monitor, manage, or treat any medical condition or disease. The user is responsible for understanding the functionality of the Seeing AI app, for assessing the conditions in which Seeing AI will be used, and for assessing the risk that use in those circumstances could result in unreasonable harm to the user or others.

Intended Users

Seeing AI is intended for use by persons who are blind or otherwise visually impaired.

Specification of Clinical Benefits and Performance Characteristics

Provides audio/haptic information to users with the aim of augmenting their understanding of the physical environment

Reusability

This device is single patient reusable. There are no requirements regarding cleaning, disinfection, or sterilization. There is no maximum number of times it can be reused.

Interoperability

Seeing AI is a mobile app, for use with the iOS and Android platforms.

Precautions

Seeing AI is not always accurate. It should not be used in situations where you could

be harmed or injured. It is not intended for use in the detection, diagnosis, monitoring, management or treatment of any medical condition or disease. Users should seek any medical advice from a physician. The user is responsible for understanding the functionality of the Seeing AI app, for assessing the conditions in which Seeing AI will be used, and for assessing the risk that use in those circumstances could result in unreasonable harm to the user or others.

Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Known Risks

The use of Seeing AI has been clinically evaluated, and has been demonstrated to be safe and effective for use as intended by the manufacturer. There are no known intolerable residual risks that users need to be aware of when using Seeing AI.

Contact Microsoft

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For all incidents, feedback, or for support with the Seeing AI application, or to request a paper copy of this manual that will be provided at no additional cost within 7 days, contact the Microsoft Disability Answer Desk for assistance: seeingai@microsoft.com

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
Holds Certificate Number:

MD 697066

and operates a Quality Management System which complies with the requirements of ISO 13485:2016 & EN ISO 13485:2016 for the following scope:

Design, development, manufacture, distribution and software maintenance of medical device software in relation to medical imaging, physiological measurements and/or analysis, and assist with sensory impairment.

For and on behalf of BSI:

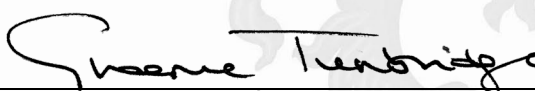Graeme Tunbridge, Senior Vice President Global Regulatory & Quality

Original Registration Date: 2019-02-03

Latest Revision Date: 2025-01-28

Effective Date: 2025-02-03

Expiry Date: 2028-02-02

Page: 1 of 1

...making excellence a habit.™